



AED Monthly Checklist

4 Oct 2025

Complete

Score	14 / 15 (93.33%)	Flagged items	0	Actions	0
-------	------------------	---------------	---	---------	---

Location of AED	Oak View, Manor View, Main St, Newark NG23 5DE, UK (53.0473372, -0.6886275)
AED Serial No.	0000137073
Checked on	4 Oct 2025 09:50 BST
Checked by	Graham Doney

AED Monthly Checklist		14 / 15 (93.33%)
Is the AED secure in its container?	Yes	
Check AED is on standby mode - green flashing light	<input checked="" type="checkbox"/>	
If AED is not on standby mode, verify AED operation		
Verify AED Operation		4 / 4 (100%)
Open AED lid / Turn on AED	<input checked="" type="checkbox"/>	
Wait for status indicator to change from RED to GREEN after five seconds	<input checked="" type="checkbox"/>	
Close AED lid / Turn off AED	<input checked="" type="checkbox"/>	
Wait for status indicator to change from RED to GREEN after five seconds	<input checked="" type="checkbox"/>	
Are the electrode pads in the AED container?	Yes	
Check expiration date of electrode pads	<input type="checkbox"/>	
Electrode Pads Expiration Date:	24 Dec 2026	
Is there an extra set of electrode pads?	Yes	
Check expiration date of AED battery	<input checked="" type="checkbox"/>	
The AED battery is expired if 4 years have passed since its installation.		
AED Battery Installation Date:	10 Jul 2023	
Check PPE/Ready Kit		4 / 4 (100%)
Mask	<input checked="" type="checkbox"/>	
Scissors	<input checked="" type="checkbox"/>	
Razor	<input checked="" type="checkbox"/>	
Gloves	<input checked="" type="checkbox"/>	
Test AED container tamper alarm	<input checked="" type="checkbox"/>	

IMPORTANT Maintenance

If electrode pads were expired, did you replace them?

N/A

Input the date the electrode pads were replaced:

1 Jun 2025

If AED battery was expired, did you replace it?

N/A

Input the date the AED battery was replaced:

10 Jul 2023

If the AED container tamper alarm was not working, did you change its battery?

N/A

Signature



Graham E Doney
4 Oct 2025 09:54 BST