



AED Monthly Checklist

16 Mar 2026

Complete

Score	15 / 15 (100%)	Flagged items	0	Actions	0
Location of AED	Oak View, Manor View, Main St, Newark NG23 5DE, UK (53.0473372, -0.6886275)				
AED Serial No.	0000137073				
Checked on	16 Mar 2026 11:41 GMT				
Checked by	Graham Doney				

AED Monthly Checklist		15 / 15 (100%)
Is the AED secure in its container?		Yes
Check AED is on standby mode - green flashing light		<input checked="" type="checkbox"/>
If AED is not on standby mode, verify AED operation		
Verify AED Operation		4 / 4 (100%)
Open AED lid / Turn on AED		<input checked="" type="checkbox"/>
Wait for status indicator to change from RED to GREEN after five seconds		<input checked="" type="checkbox"/>
Close AED lid / Turn off AED		<input checked="" type="checkbox"/>
Wait for status indicator to change from RED to GREEN after five seconds		<input checked="" type="checkbox"/>
Are the electrode pads in the AED container?		Yes
Check expiration date of electrode pads		<input checked="" type="checkbox"/>
Electrode Pads Expiration Date:		24 Dec 2026
Is there an extra set of electrode pads?		Yes
Check expiration date of AED battery		<input checked="" type="checkbox"/>
The AED battery is expired if 4 years have passed since its installation.		
AED Battery Installation Date:		10 Jul 2023
Check PPE/Ready Kit		4 / 4 (100%)
Mask		<input checked="" type="checkbox"/>
Scissors		<input checked="" type="checkbox"/>
Razor		<input checked="" type="checkbox"/>
Gloves		<input checked="" type="checkbox"/>
Test AED container tamper alarm		<input checked="" type="checkbox"/>

IMPORTANT Maintenance

If electrode pads were expired, did you replace them?

N/A

Input the date the electrode pads were replaced:

1 Jun 2025

If AED battery was expired, did you replace it?

N/A

Input the date the AED battery was replaced:

10 Jul 2023

If the AED container tamper alarm was not working, did you change its battery?

N/A

Signature



Graham E Doney
16 Mar 2026 11:48 GMT